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General description: Denture base polymer material Polirepar S is intended for the repairs of dentures by using the cold curing technique. It is suitable for patients of any age with complete or partial edentulism, who need to restore functions of missing teeth (mastication, phonetics) and also to provide an aesthetic appearance of the patient.

Material is supplied in form of liquid and powder that shall always be used together by the dental health care professional (dental technician).

Material characteristic	Specification	Method according to
Flexural strength	> 60 MPa	EN ISO 2795-1
Flexural modulus	> 1500 MPa	EN ISO 2795-1
Sorption	< 32 µg/mm³	EN ISO 2795-1
Solubility	< 8 µg/mm³	EN ISO 2795-1
Residual monomer	< 4,5 %	EN ISO 2795-1

Variants: Available in various colours and quantities. Consult your distributor.
Indication: Edentulism, partial edentulism
Patient group: Patients of any age with complete or partial edentulism.
Composition: Powder: PMMA based powder, benzoyl peroxide, pigments
Liquid: Methyl methacrylate, dimethacrylate, activator

Contraindications: Do not use it in the case of known allergy/hypersensitivity to acrylates or any other component. Safety data sheet available on the web page.

Undesirable side effects: In individual cases, local allergic reactions to polymethylmethacrylate materials have been reported. They may present as oral mucosal irritation, redness, burning sensation and/or swelling. In the case that any serious incident has occurred in relation to the device, it should be reported to the manufacturer (Polident d.o.o., vigilanca@polident.si) and the competent authority of the Member State, where the user is established.

DIRECTIONS FOR USE:

Do not use Polirepar S for direct relining of the denture base. Visually inspect the liquid. If you notice any sediment or gel do not use the liquid. Discard it in accordance with local legislation. We recommend temperature conditioning of the liquid and powder components at the room temperature before use.

Preparation: Coat the plaster model with the isolating fluid. Roughen the areas of the prosthesis that require repair and apply the Polirepar S liquid to those areas.

Mixing and casting: The recommended mixing volume ratio is 1,7-2,0 parts of powder to 1 part liquid. Place the liquid into a mixing jar and gradually add the powder. Mix the liquid and the powder with a spatula for about 30 sec. Cover the mixing jar and put it aside for approximately 3 minutes. After that, the mixture is ready and can be processed at room temperature (approx. 22°C) for about 4 more minutes. The working time depends on the temperature of the liquid and the powder and the room temperature.

Polymerisation: Insert the prepared model in the hot pressure pot. The conditions of polymerisation are as follows: **water temperature between 40-45°C, a pressure of 6 bar and a polymerisation time of 20 minutes.** **Trimming:** After removing the plaster, trim and polish denture following common dental laboratory procedures.

NOTE! Dental restorations made of acrylics always contain a small proportion of residual monomer. The residual monomer content is below 4,5% if the polymerisation is performed according to the instructions for use. For further free monomer reduction, it is recommended to store a restoration in water for at least 24 hours. If this step is not possible, advise the patient to place the appliance in water whenever it is not being worn. Not following the instructions may cause residual monomer levels above safety limit, poor physical properties, or biofilm formation due to inadequate surface quality.

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Information to be provided to the dentist: In accordance with standard dental medicine practices, dentist must clean medical device before inserting it into the oral cavity. The dentist must provide the patient with instructions for adequate cleaning of the medical device and follow up the patient at regular appointments to assess the condition of the medical device. We recommend cleaning with soft brush and non-abrasive toothpaste. Non-compliance with these instructions may result in increased biofilm formation or discoloration of the denture base material.
Storage: Store the liquid at temperatures up to 25 °C, keep protected from sunlight. Store powder at temperature up to 30°C, keep protected from sunlight, keep dry.

General warnings for the dental technician: Do not use material after expiry date. Expiry date is printed on the packaging. Keep out of reach of children. Keep the liquid and powder in the original packaging only. Do not use if the primary packaging is damaged. Avoid direct contact between skin and uncured material as well as inhalation of the vapours. It is necessary to wash your hands after handling the uncured material. Close containers immediately after use. Ensure efficient ventilation of the workspace. Ensure sufficient extraction of the particles from the workplace during the finishing steps (trimming, grinding, polishing). Wear suitable personal protective equipment (PPE) during all procedures. If no technical protective measures are taken, irritation of the eyes, skin and respiratory organs may occur during the handling with uncured material and finishing procedures. Consult the Safety Data Sheet (SDS) for proper personal protective equipment (available on the website https://www.polident.si). Do not store the powder at temperatures above 50°C for long period of time.

Warnings for liquid component: Liquid is highly flammable. Irritating to eyes, respiratory system and skin. May cause sensitization by skin contact (allergic skin reaction). Keep away from sparks, open flames, and other sources of ignition. Do not smoke. Avoid breathing vapours. Bottles must be tightly closed and stored in well-ventilated place. Do not get in eyes, on skin, or on clothing. Wear protective gloves/protective clothing/eye protection. Take off immediately all contaminated clothing. Rinse skin with water or shower. Do not empty into drains. Take precautionary measures against static discharges.

Disposal: Do not empty unused product in drainage systems! The product must not be disposed of with municipal waste. Dispose in accordance with the local and/or national regulations.

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